MOBILES STRAHLENSCHUTZ-STATIV

STS-8000/760
STS-8000/1000

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March 2019

1. Specifications

- Mobile Holder for lead-acrylic protective shield with lead equivalent of Pb 0.5 mm (Manufacturer: MAVIG GmbH. MAVIG is a registered trademark)
- Shield dimensions: 60 x 76 cm with a cutout for the patient
- Elevation of shield center: 90 to 150 cm
- Ergonomically designed: Weight counterbalanced operation
- Length of the radial arm: 76 cm or 100 cm
- Stainless steel casters are electrically conducting, 10 cm diameter
- Base dimensions: 54 x 48 cm
- Total height: 122 cm
- Total weight: 50 kg

2. Intended Use

Protective device against X-ray radiation for medical personnel (and any person in the radiation area). Radiation reduction of the shield with lead equ. of 0.5mm

The device is not intended to protect patients and should not have patient contact.. The shield has to be placed between the x-ray source and the user.

3. Safety information and inappropriate Use

- Follow these directions before use.
- Operation by authorized personnel only
- Move carefully to avoid overturning on an uneven floor
- Always apply the caster brakes.
- Do not apply any additional load or device.
- Do not use the device if it has an obvious defect.
- Devices with any damage of the shield (e.g. fissure, splinter or defective surface) reduces the radiation protection and may not be used.

4. Cleaning and Disinfection

Only use a cloth dampened with mild suds and do not use aggressive cleaning agents (solvents or similar).

Warning! Aggressive or abrasive agents or solvents may damage the shield's surface!

When used in sterile areas surface disinfection is recommended .
Periodically cleaning of the casters rubber is recommended.

5. Maintenance

Maintenance is not necessary.
The device may only opened by trained service engineers.

6. Guarantee

The manufacturer does not take on any responsibility for damages or bodily injuries and guarantee is not valid in following circumstances:

- non observation of the guidelines as laid out in the operating instructions
- use outside of the described applications
- alteration to or opening of the device
- mechanical damage and damage by exposure to improper substances, liquids
- natural wear and tear
- inappropriate erection or installation
- overloading the device
- use of the device with an obvious defect

7. Disposal

The operating company shall dispose of the packaging and the device in compliance with the valid national or regional law of the operating location.

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Product name:

*Holder for Radiation
Protection Device*

Type / *type:*

STS-8000/760
STS-8000/1000

Seriennummer:
Serial number:

Laut Typenschild
According to the type label

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I declare under sole responsibility that the product above is compliant with directive 2006/42/EC.

Technische Dokumentation bei:
Technical documentation at:

Dr. Franz Merksa
Rudolf Kassner-Gasse 36
1190 Vienna, Austria

Vienna/Austria, 2024-03-04

Dr. Franz Merksa